


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/2 2993461

HEAD SIDE / 2030			
Scale: 1:1	Drafted	Verified	Confirmed
Date	13/01/2017	20/11/2023	20/11/2023
Initials	KR	SD	PJ

Plan No.  
EP00228/G

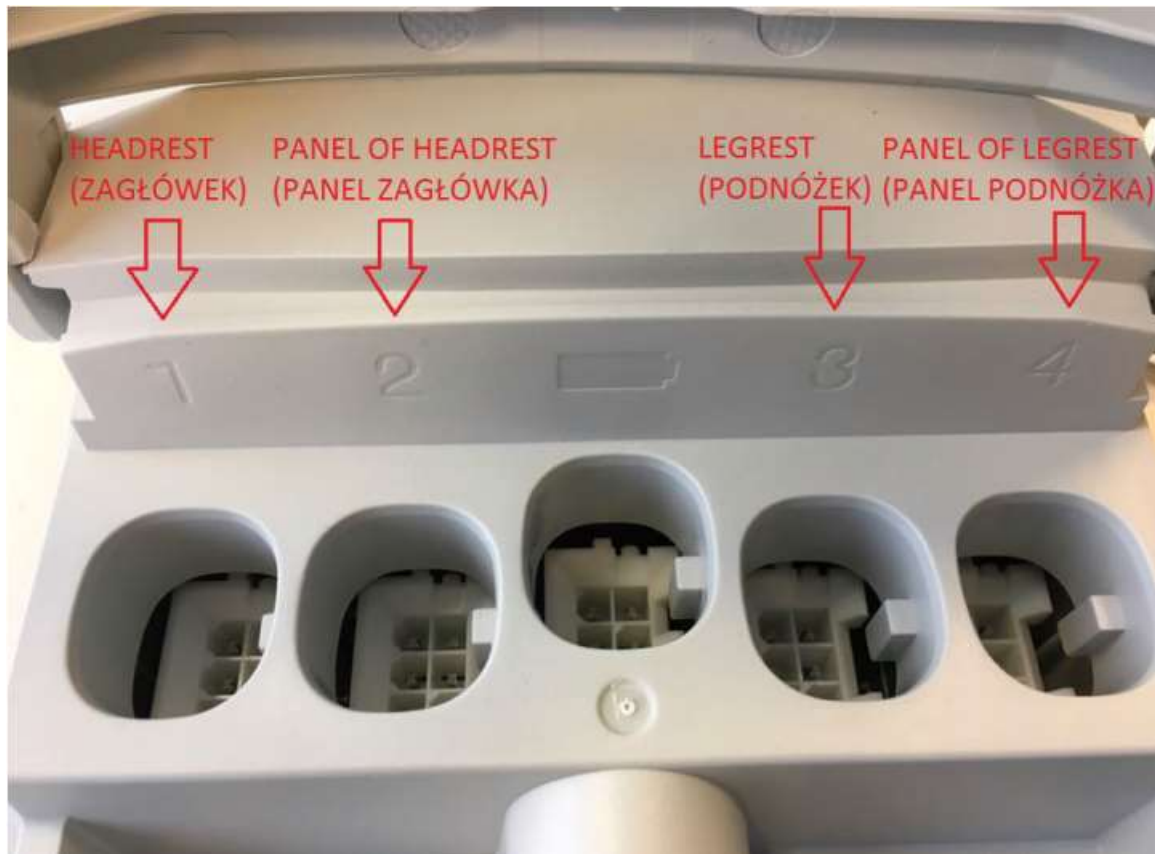

Bill of Material			
No.	Plan	Article	Quantity
1	YP05416 DP	2709908	1
2	YP05413	2710008	1
3	X001938	1991532	12
4	X002618	1994185	6
5	M10 DIN 934	1001672	2
6	D10,5 DIN 125	1001557	4
7	X002720	1551600	2
8	D25	1001608	2
9	M5x23	2990016	2
10	X002629	1994230	2
11	XP00076	GP00300	2
12	M10 DIN 985	1001683	2
13	X001336	1400800	4
14	M10	1009994	2
15	D10x50	1998885	2
16	Motor LA 40	1912899	1
17	40x20	1001615	2
18	---	H016007	12
19	---	1912902	3
20	---	1912903	1
21	---	1902971	1
22	---	1913262	1
23	---	1920271	1
24	---	U00046A	1
25	Cable number 1	1995835	1
26	Cable number 2	1995836	1
27	Cable number 3	1995837	1
28	Cable number 4	1995838	1
29	---	1912898	1
30	YP06245	2795908	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/2

2993461

HEAD SIDE / 2030

Plan No.

EP00228 / G

Scale: 1:6	Drafted	Verified	Confirmed
Date	13/01/2017	20/11/2023	20/11/2023
Initials	KR	DB	TK

